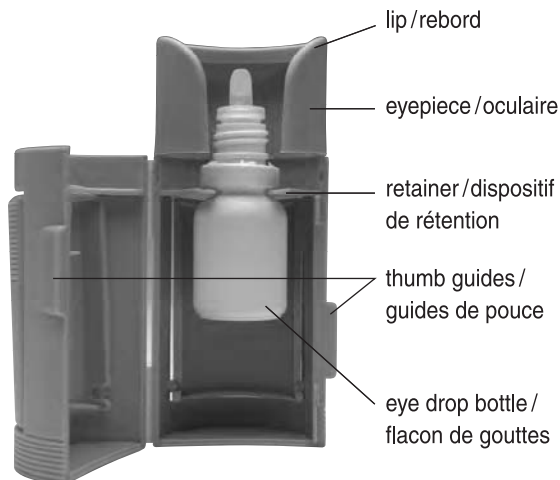
# Opticare®

## Instructions

Opticare eye drop dispenser targets the eye and squeezes the bottle

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2. Place the bottle inside Opticare so that the retainer fits firmly around the base of the bottle neck.

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2a. **Some** bottles (e.g. Travatan, Xalatan) require a bottle collar to ensure a secure fit. Slide collar (supplied) around the base of the bottle neck.

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2b. Place bottle inside Opticare as per step 2. The optional collar sits on top of the retainer and the base of the bottle neck sits below the retainer.

3. Close Opticare by clicking sides together securely. Open eyepiece to remove bottle cap before use.

4. Use finger to pull down lower eye lid. Tilt head back and rest lip of eyepiece gently against upper eye socket. Gently squeeze the side panels to deliver drop(s) into the eye. Replace bottle cap after use.

*To ensure proper hygienic care, always wash Opticare in hot water before installing a new eye drop bottle.*

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**Australia/Australie**

Intelligent Health Systems  
Factory 2, 8-12 Coora Road  
South Oakleigh, Victoria 3166  
www.Intelligenthealthsystems.com.au

**Canada**

McArthur Medical Sales Inc.  
1846 5th Concession West  
Rockton, Ontario L0R 1X0  
www.mcarthurmedical.com

Opticare® is available on prescription in the UK.  
Opticare® may also be purchased at your pharmacy.  
Ask your pharmacist for PIP Code 033-3369.

UK Patent 2216802  
UK Registered Design 2014524  
US Patent D 345603  
Opticare® is a Registered Trademark

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**Cameron Graham Ltd**